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EN **INSTRUCTIONS FOR USE****DESCRIPTION**

The present instructions for use describe the normal use of

- 20232-003 LAP tubing set IES 3 with trumpet valve 3m
- 20232-006 LAP tubing set IES 3 with trumpet valve 5m
- 20232-010 LAP tubing set IES 3 without valve 3m
- 20232-011 LAP tubing set IES 3 without valve 5m

INDICATIONS

The product item numbers 20232-003, 20232-006, 20232-010 and 20232-011 are intended:

For use with the IES 3 Smoke Evacuation Unit.
To be connected with Luer Lock connectors to laparoscopic trocars
To evacuate smoke and aerosol-laden air during surgical interventions
Ces produits sont intended for single use only.

KONTRAINDICATIONS

These products are not suitable for the transfer of blood, blood derivatives, fluids or medications. Do not connect the product to a low vacuum drainage.

PRECAUTIONS FOR USE

These products should be used after an assessment by the surgeon and under his or her own responsibility. These products may be used only by trained medical staff, which has been shown how to use them according to the instructions for use.

Please read all information carefully. These instructions for use do not replace the user manual of the IES 3 Smoke Evacuation Unit used! Read the user manual of the IES 3 Smoke Evacuation Unit and ask Erbe Elektromedizin GmbH or your distributor in case of doubt!

CONTRAINDICATIONS FOR USE

Peel open the packaging following aseptic procedures.

1. Check the use-by date

The use-by date is indicated on the packaging. Do not use this product if the use-by date has been exceeded!

2. Check and open the packaging

Only use this product if the packaging is unopened and undamaged. If the packaging is open or damaged, the product is no longer sterile and must not be used.

Use aseptic techniques when removing the product from the packaging!

3. Verify the product

Check the product for damage before using. If damaged, do not use the product!

4. Connect the product

To install the product please read the user manual of IES 3 Smoke Evacuation Unit.

5. Adjust evacuation intensity

The desired evacuation intensity is set on the IES 3 itself (20232-010/20232-011) and/or influenced via the valve on the tubing (20232-003 and 20232-006).

CONDITIONS OF USE AND STORAGE:

The product should be stored and transported in a normal environment, i.e. away from extreme temperatures and humidity. Do not use the product if the sterile packaging is damaged and/or deteriorated. Single-use product: do not reuse, reprocess or sterilize. This could compromise integrity of the product and/or cause malfunctions potentially leading to injury, illness or death of the patient. In addition, there is a risk of contaminating the product, which could cause an infection in the patient or cross-infection with another patient which could result in injury, illness or death. In this case the manufacturer will not be held liable. After use, discard the product and packaging in compliance with hospital, government and/or local authority policies.

LIABILITY AND WARRANTY:

PROMEPLA and affiliates declare to be the first purchaser of the product that reasonable care was taken in the design and manufacture of this product. If damage is suspected, please contact PROMEPLA and its representatives. The present limited warranty dispenses all other warranties, express or implied, including but not limited to any implied warranties concerning merchantability or suitability for specific purposes. This limited guarantee ensures that the responsibility of PROMEPLA and affiliated companies will be limited to the replacement of defective products. Under no circumstances shall PROMEPLA and affiliates be held liable for direct, indirect, incidental and/or consequential damages of any kind arising from the use and/or handling of this product.

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IT **PROMEPLA S.A.M**